



Do you want to use the Setup Walkthrough?

The Walkthrough is a step by step process that will take you through the whole setup.

Use Walkthrough

Go to Home Page

Arm



Peripherals



Room



Draping



Connections

Approach Angle



Base Position



Arm Position

Choose the approach angle.

Image of Modus V arm in vertical configuration

Vertical

For approaches or trajectories that are vertical or close-to-vertical.

Image of Modus V arm in horizontal configuration

Horizontal

For approaches or trajectories that are lateral or close-to-horizontal.

"Don't Know" image

Don't Know

Choose this if you don't know the approach angle.



Arm



Peripherals



Room



Draping



Connections

Approach Angle



Base Position



Arm Position

Examine the cable. Ensure that it is not wrapped around the arm. Pay special attention to the cable bundle before moving the arm.

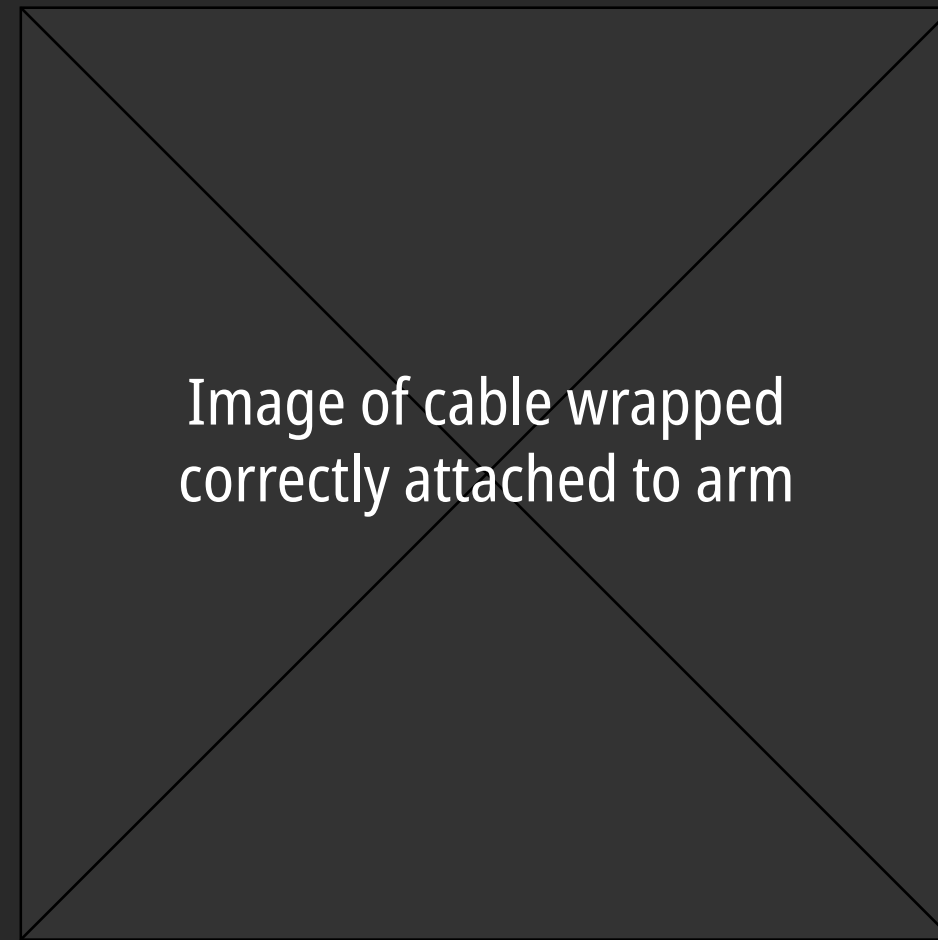


Image of cable wrapped correctly attached to arm

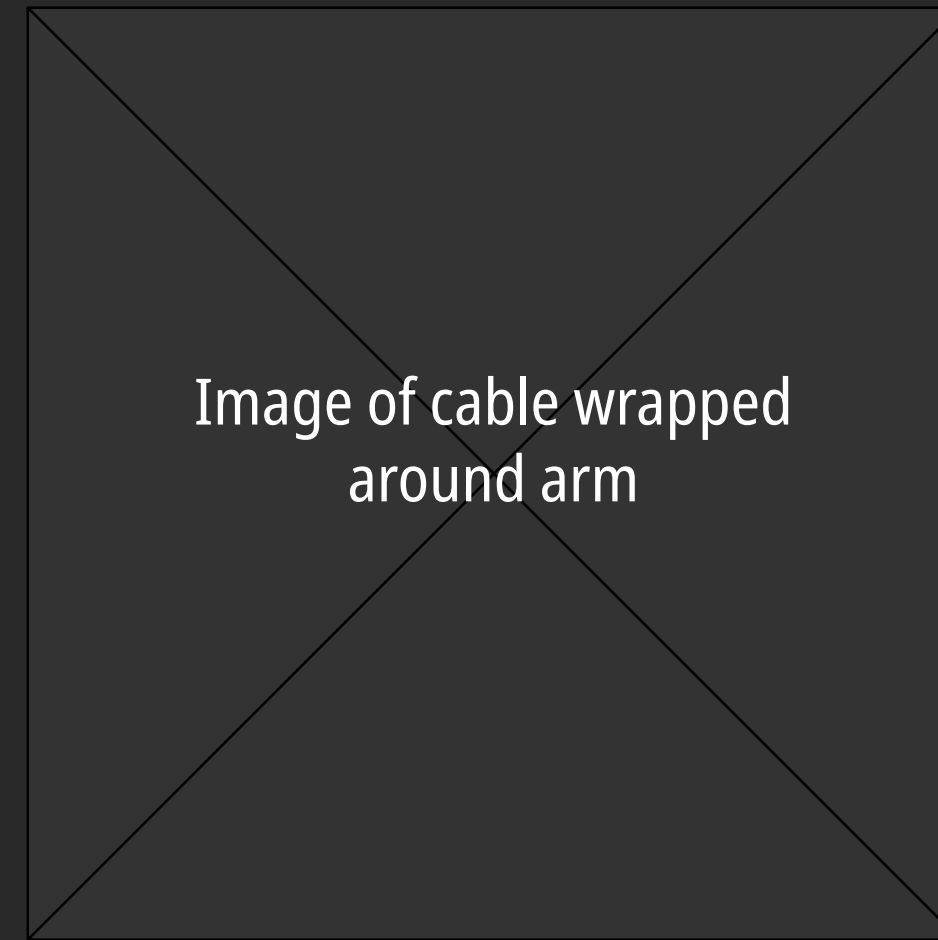


Image of cable wrapped around arm



Got it

Arm



Peripherals



Room



Draping



Connections

Approach Angle



Base Position



Arm Position

Choose the location of the Modus V base.



Image of surgeon with Modus V on his/her right

Right

Modus V is on the surgeon's right.

Image of surgeon with Modus V on his/her left

Left

Modus V is on the surgeon's left.

"Don't Know" image

Don't Know

Choose this if you don't know the location.





Arm



Peripherals



Room



Draping



Connections

Approach Angle




Base Position



Arm Position



Caution! The arm joints are not in an optimal configuration.
You can proceed but there may be collisions.
An Arm Flip is needed to fix the problem.

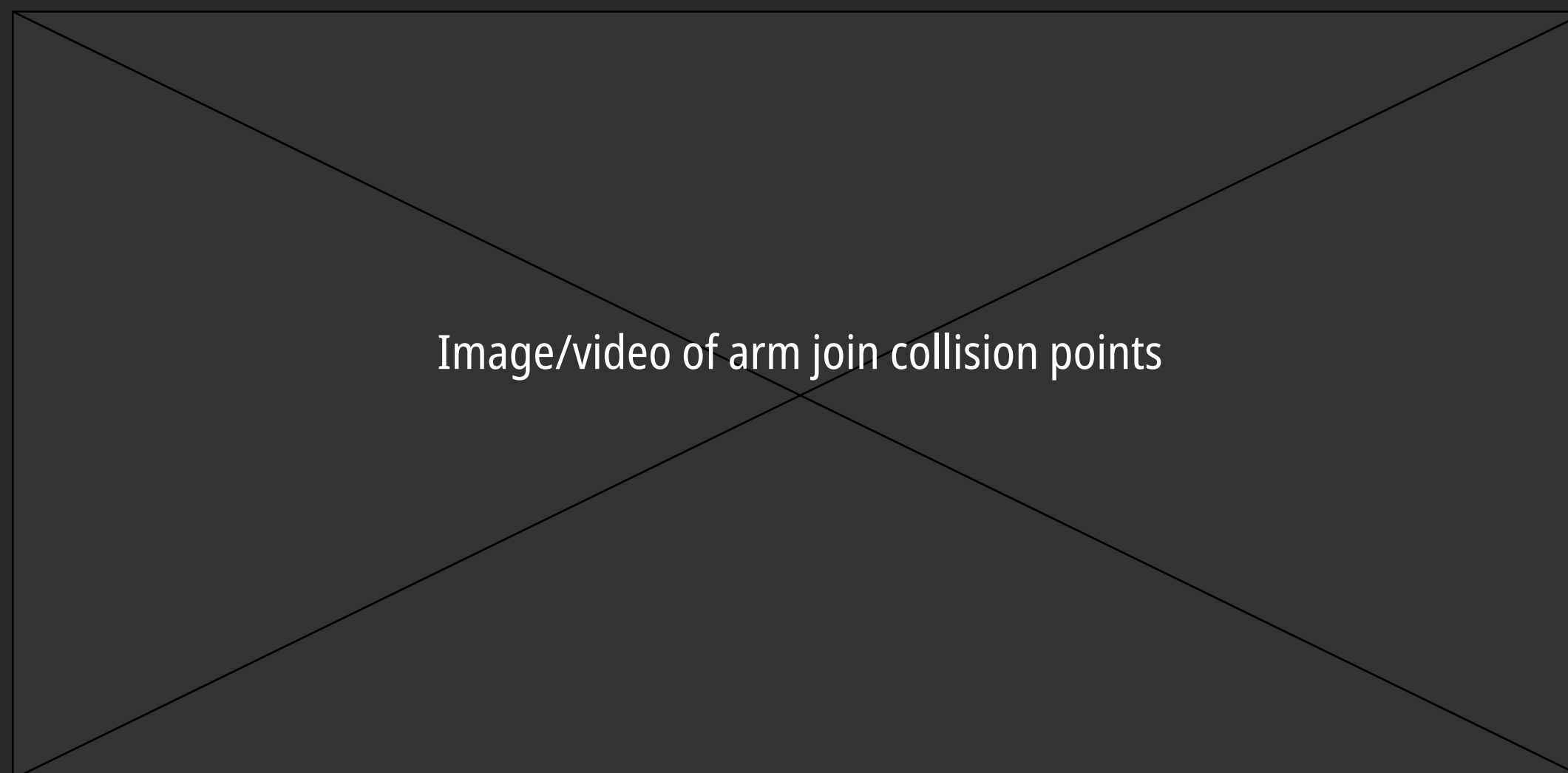

Image/video of arm join collision points



BACK



NEXT

Continue without Arm Flip

Start Arm Flip Walkthrough



Arm Flip Wizard

Arm flip step thumbnail	Arm flip step thumbnail
Arm flip step thumbnail	Arm flip step thumbnail
Arm flip step thumbnail	Arm flip step thumbnail

Arm flip detail image/video

Close

Go to Home Page

Arm



Peripherals



Room



Draping



Connections

Approach Angle




Base Position



Arm Position

Move the arm to a position suitable for the case.
The position will be saved when you go the next step in the walkthrough.

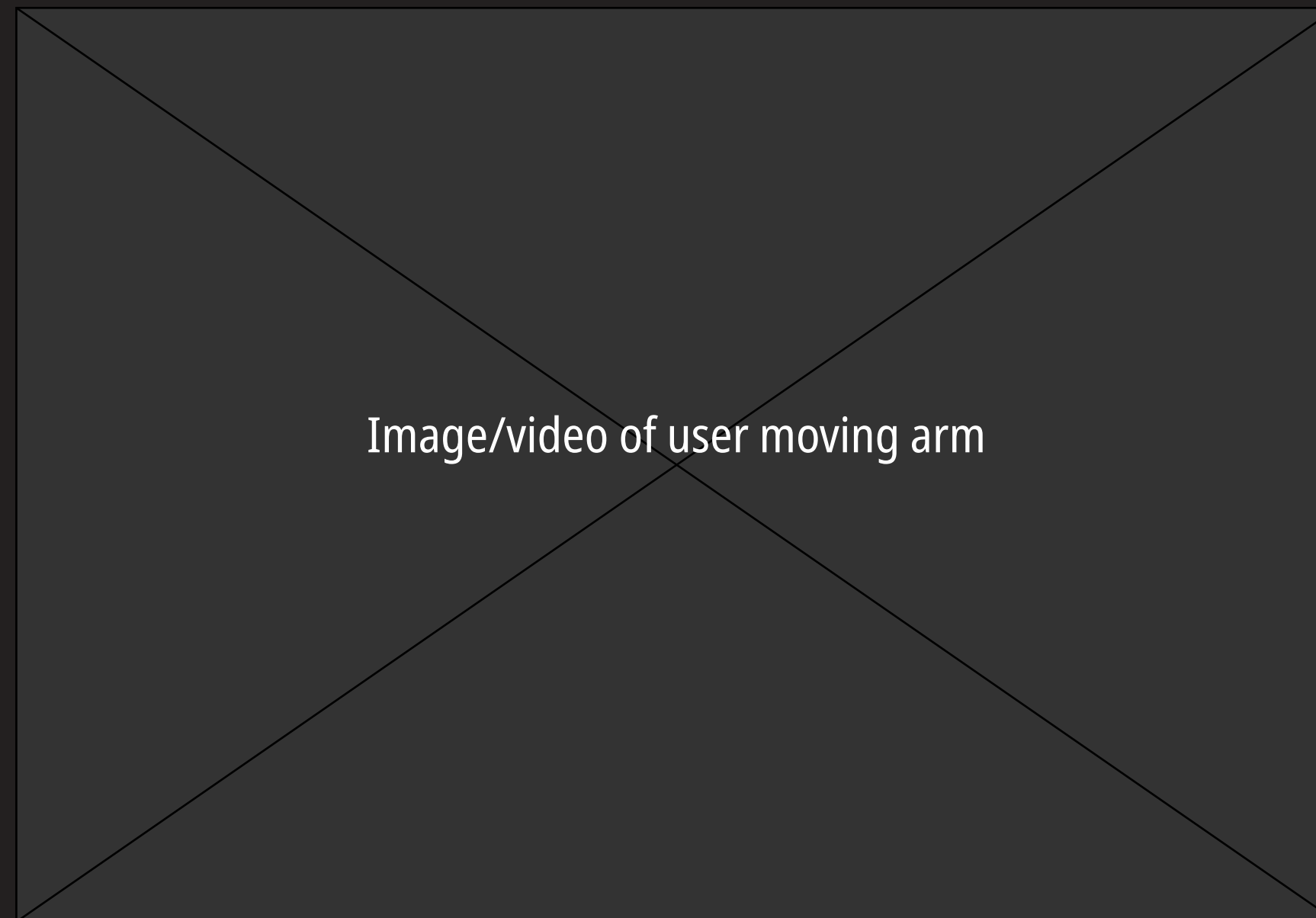

Arm



Peripherals



Room



Draping



Connections

Tools



Components



Pedal

Choose the tracked tools for the case.



Image of pointer

Pointer



Image of port

Port



Image of suction tool 1

Suction 1

Image of suction tool 2

Suction 2



Arm



Peripherals



Room



Draping



Connections

Tools



Components



Pedal

Choose the tracked tools for the case.



Image of pointer

Image of port

Image of suction tool 1

Image of suction tool 2

BACK

NEXT

Pointer

Port

Suction 1

Suction 2

Arm



Peripherals



Room



Draping



Connections

Tools



Components



Pedal

Choose the components for the case.

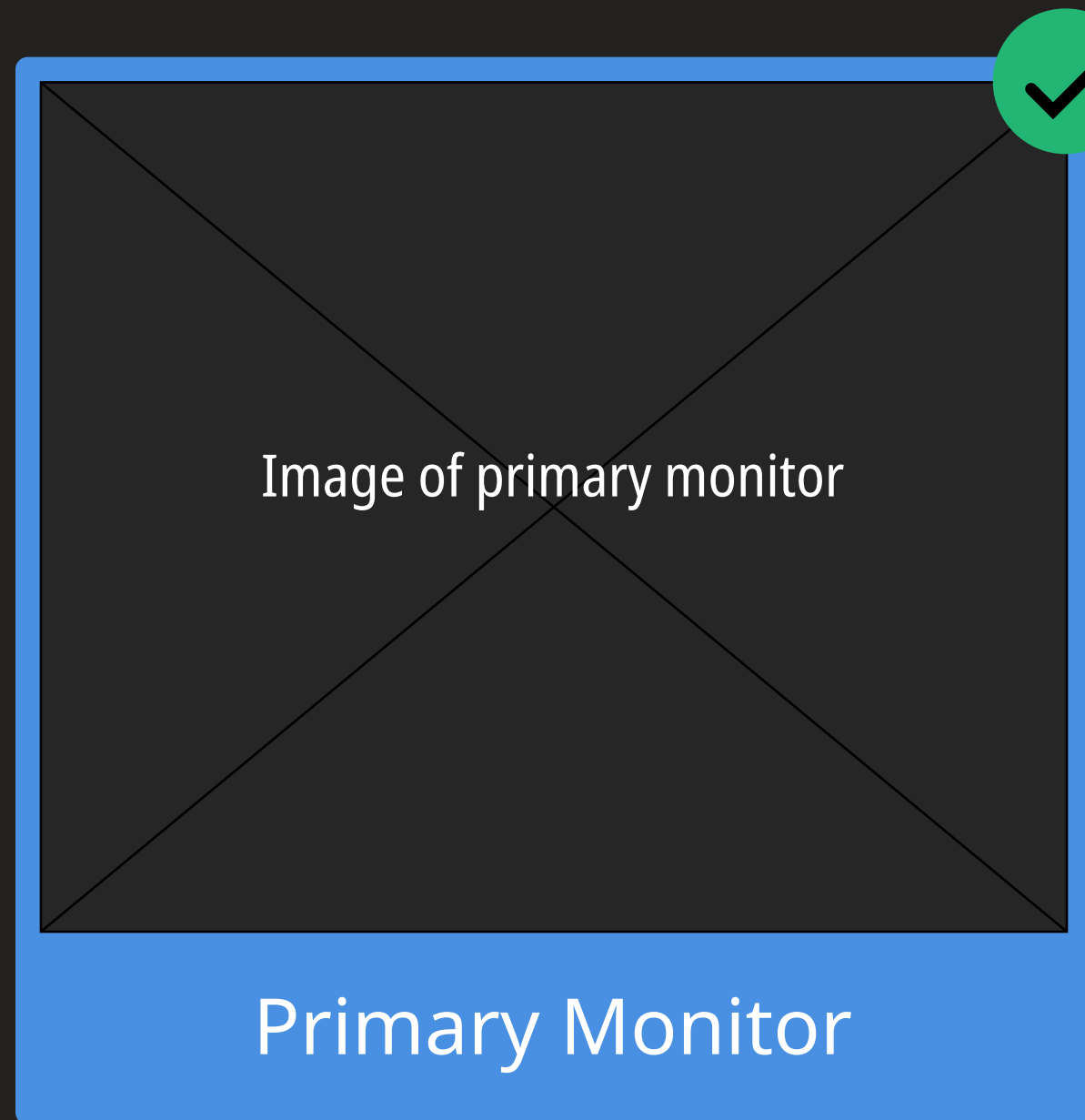


Image of primary monitor

Primary Monitor

A blue-bordered card with a dark grey background. It contains a placeholder image for a primary monitor with a diagonal cross. A green checkmark icon is in the top right corner. Below the card is a blue bar with the text "Primary Monitor".

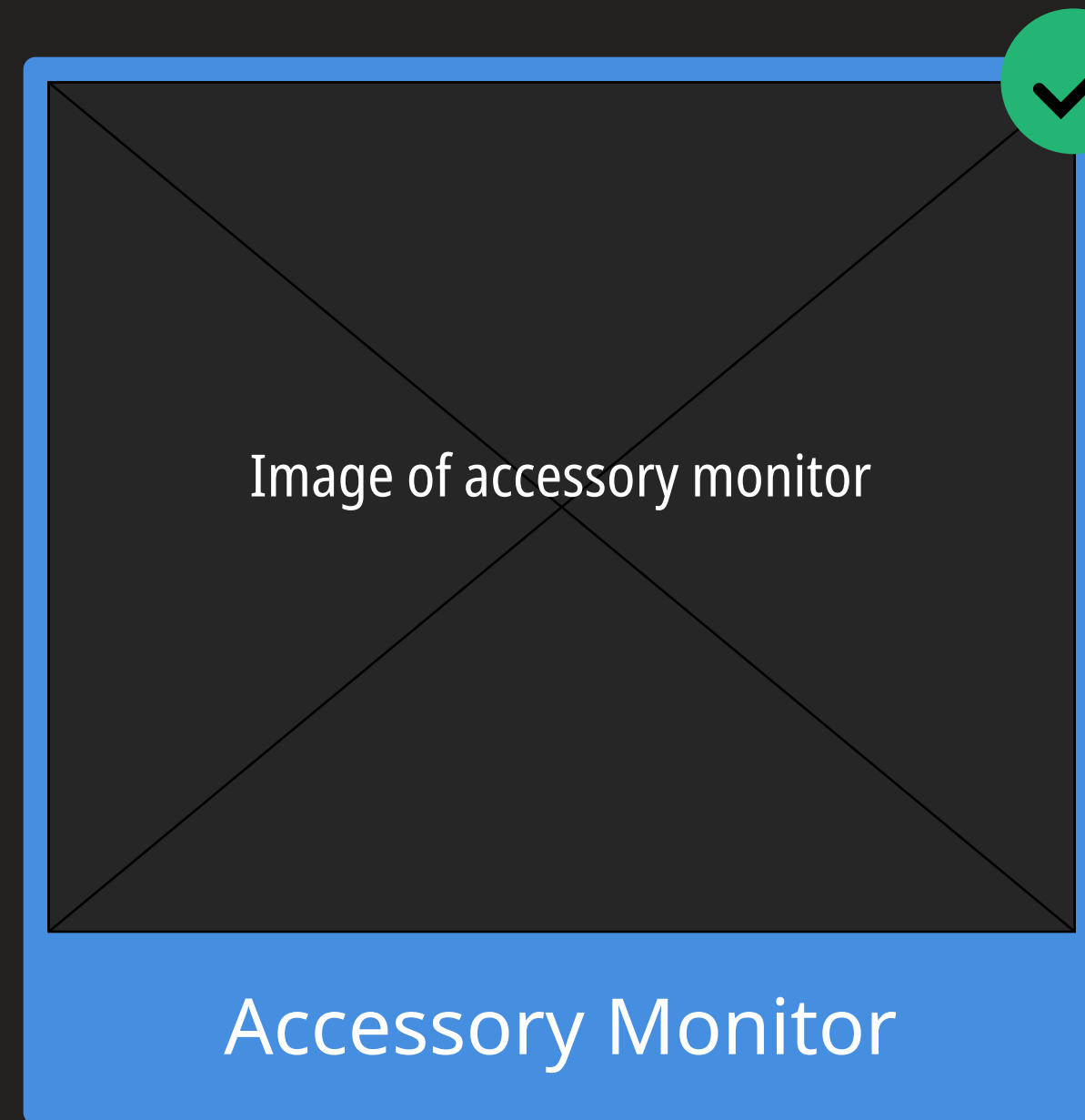


Image of accessory monitor

Accessory Monitor

A blue-bordered card with a dark grey background. It contains a placeholder image for an accessory monitor with a diagonal cross. A green checkmark icon is in the top right corner. Below the card is a blue bar with the text "Accessory Monitor".

Optional

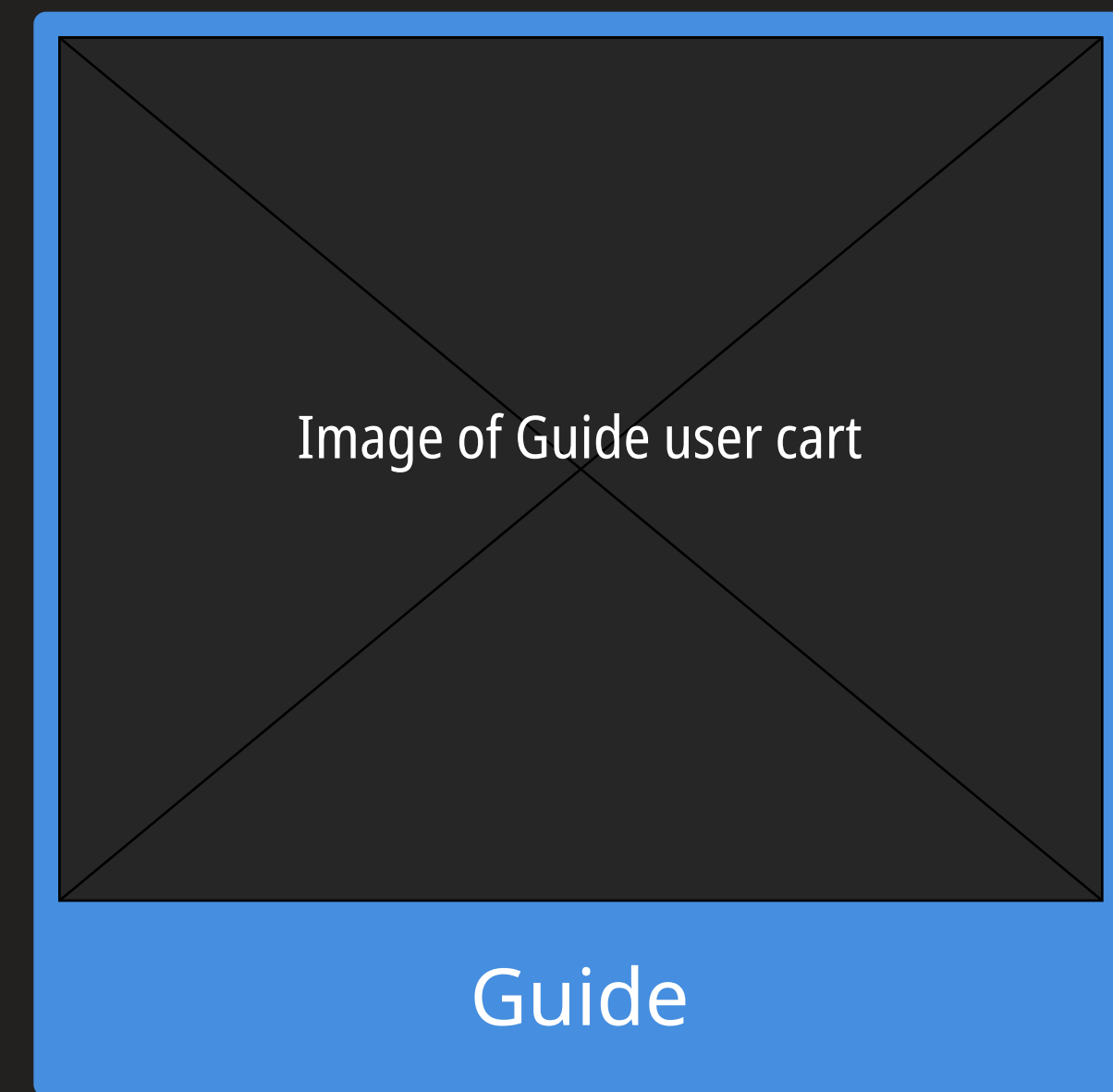


Image of Guide user cart

Guide

A blue-bordered card with a dark grey background. It contains a placeholder image for a Guide user cart with a diagonal cross. Below the card is a blue bar with the text "Guide".

Optional



Arm



Peripherals



Room



Draping



Connections

Tools



Components



Pedal

Choose the pedal for the case.



Image of multi-function pedal

Multi-function

Image of single-function pedal

Single-function



Arm



Peripherals



Room



Draping



Connections

Room Setup



Tracker Placement

Consult with the surgeon and position the components in the operating room.

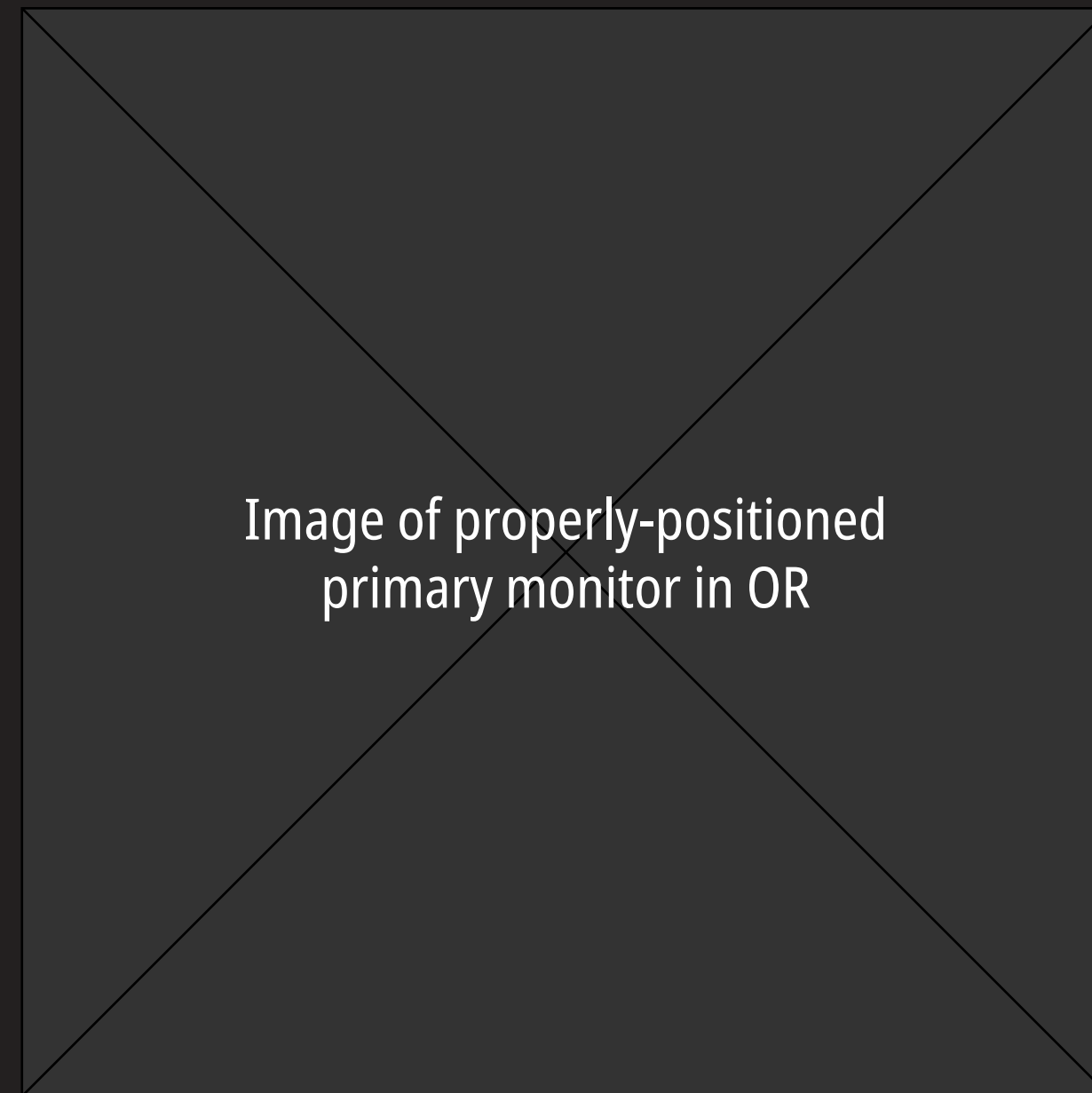


Image of properly-positioned primary monitor in OR

The Primary Monitor must be positioned so the surgeon can see it easily.



Image of properly-positioned accessory monitor in OR

The Accessory Monitor must be positioned so the assisting surgeon or others can see it easily.



Arm



Peripherals



Room



Draping




Connections

Room Setup



Tracker Placement

Use the Tracking Camera volume to position tracked tools and end effector.

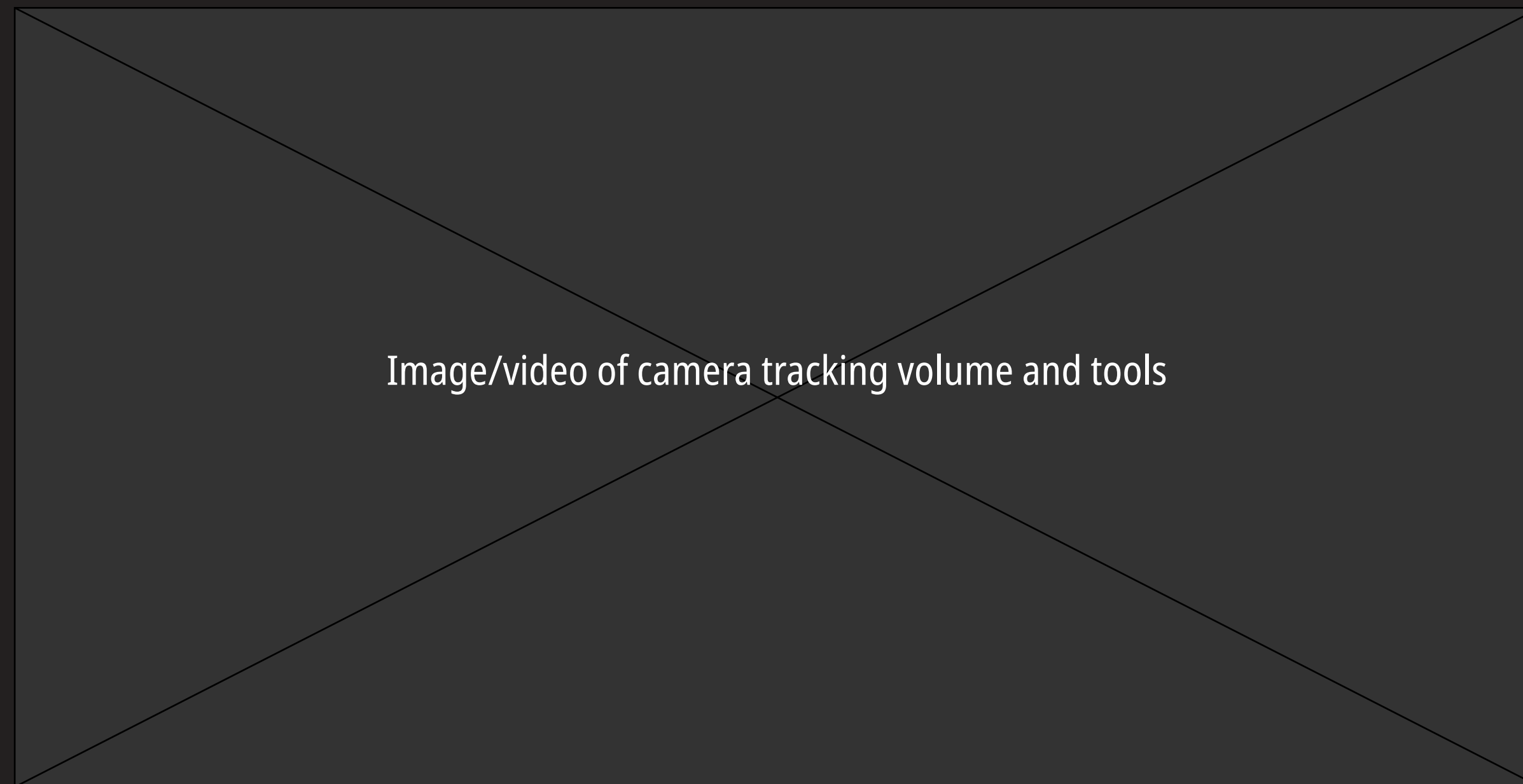

BACK



NEXT

The tracking camera must have an unobstructed view of the end effector and the tracked tools.
The end effector and tracked tools must be within the tracking camera's field of view.

Arm



Peripherals



Room



Draping



Connections

Arm




Skirt



Touchscreen

If necessary, move Modus V to a suitable area for draping.
Watch the video to see how to drape the arm.

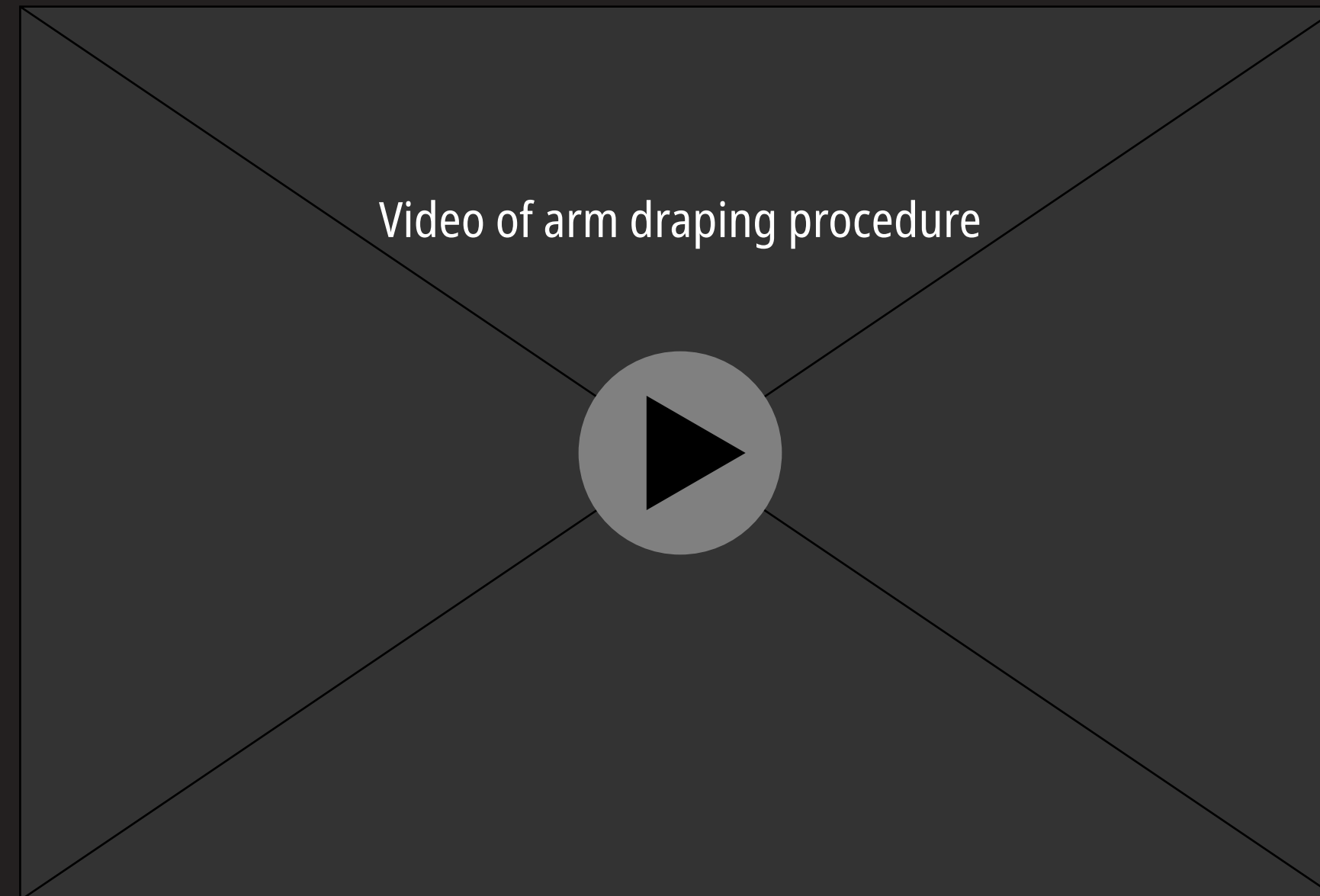

Arm



Peripherals



Room



Draping



Connections

Arm




Skirt



Touchscreen

If necessary, move Modus V to a suitable area for draping.
Watch the video to see how to drape the skirt.

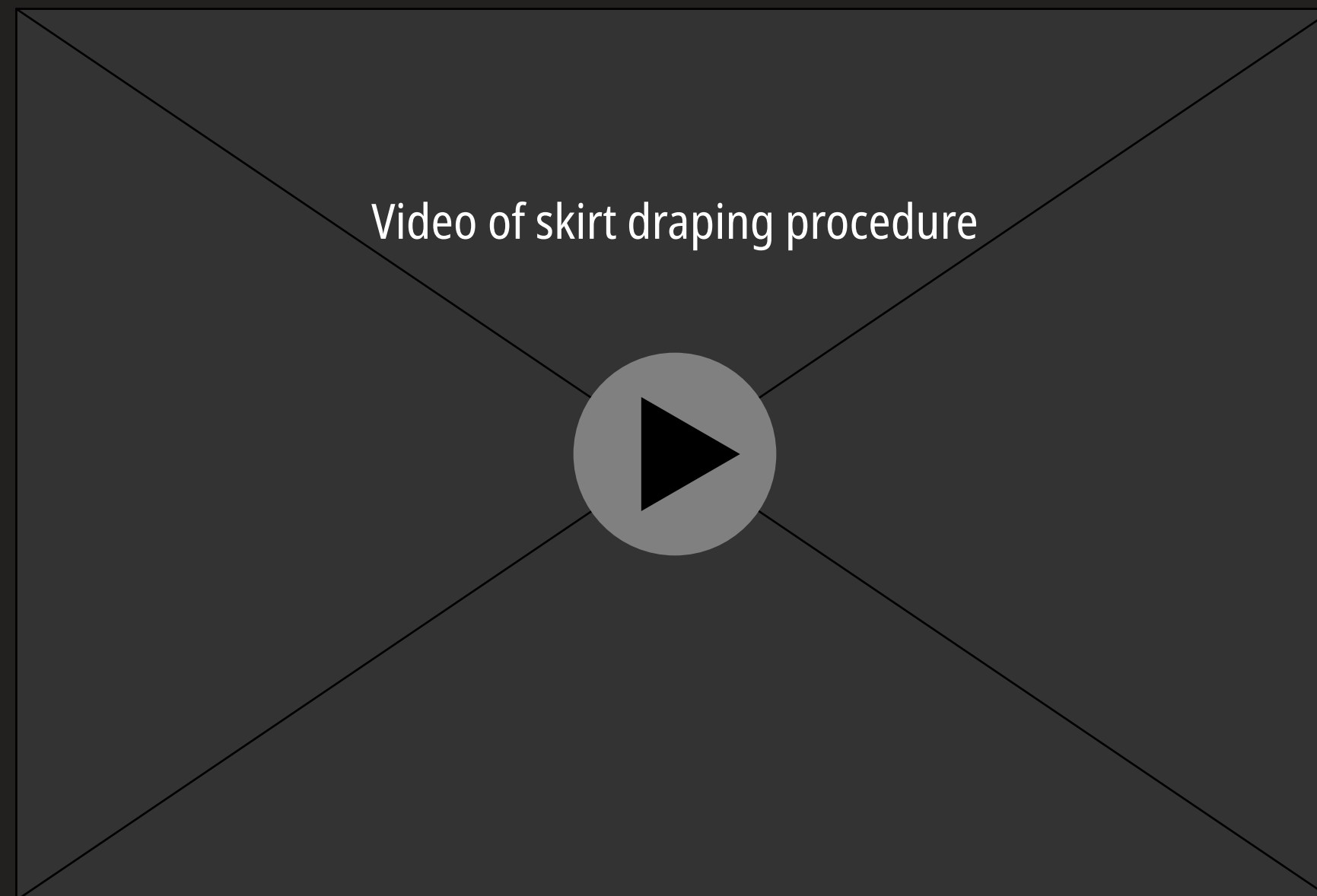

Arm



Peripherals



Room



Draping



Connections

Arm




Skirt



Touchscreen

If necessary, move Modus V to a suitable area for draping.
Watch the video to see how to drape the touchscreen. This step is optional.

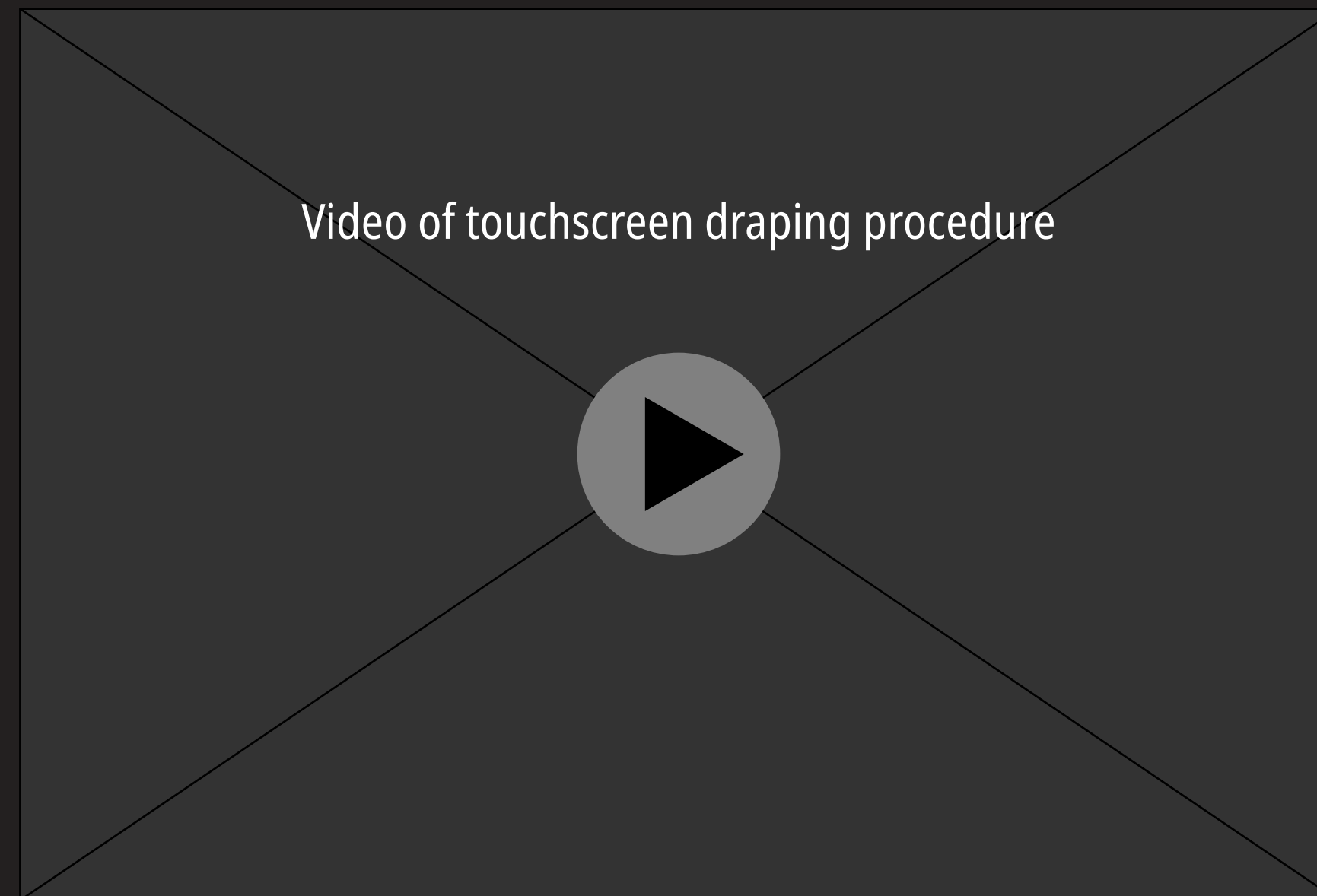

Arm



Peripherals



Room



Draping



Connections

Primary Monitor



Accessory Monitor



Guide

Connect the Primary Monitor as shown.



Arm



Peripherals



Room



Draping



Connections

Primary Monitor




Accessory Monitor



Guide

Connect the Accessory Monitor as shown.

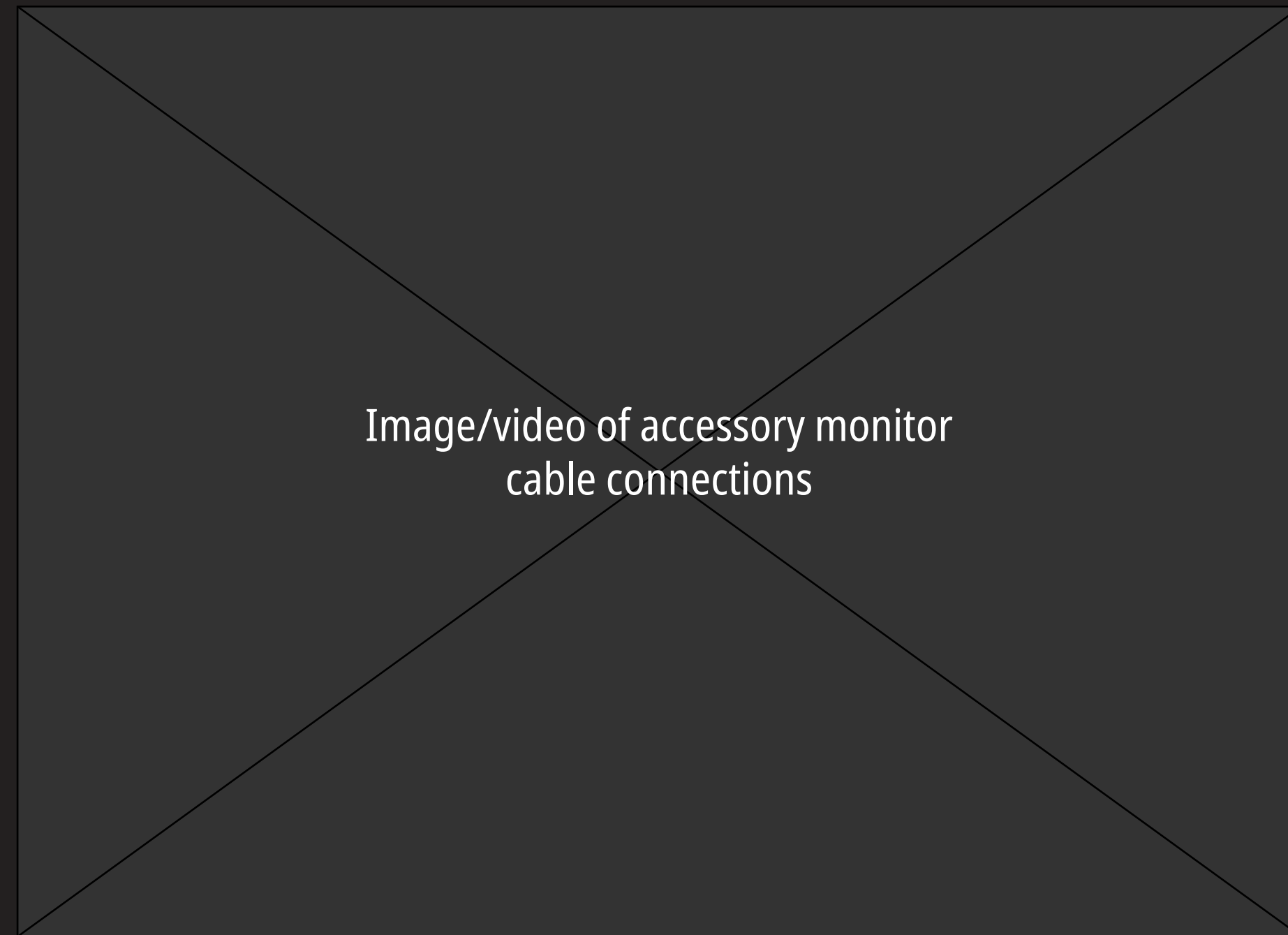

Arm



Peripherals



Room



Draping



Connections

Primary Monitor



Accessory Monitor



Guide

Connect the Guide Operator Cart as shown.





Do you want to use the Wrap-Up Walkthrough?

This Walkthrough will show how to clean and shut down Modus V.

Use Walkthrough

Go to Home Page

Move



Clean



Shutdown

Move Modus V




Disconnect Cables



Engage Brakes

Move Modus V away from the patient.

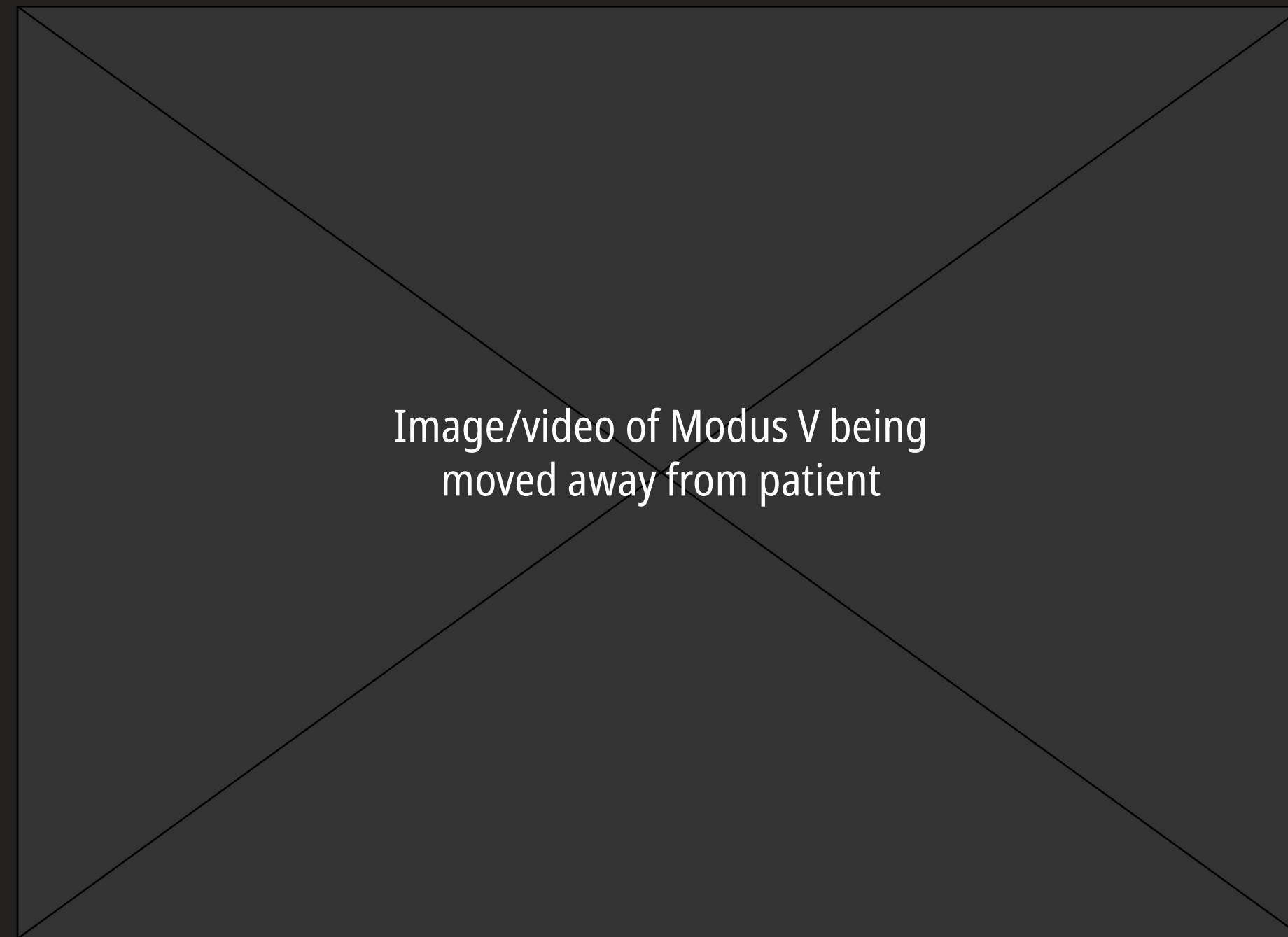

NEXT



Move



Clean



Shutdown

Move Modus V




Disconnect Cables



Engage Brakes

Disconnect all cables.

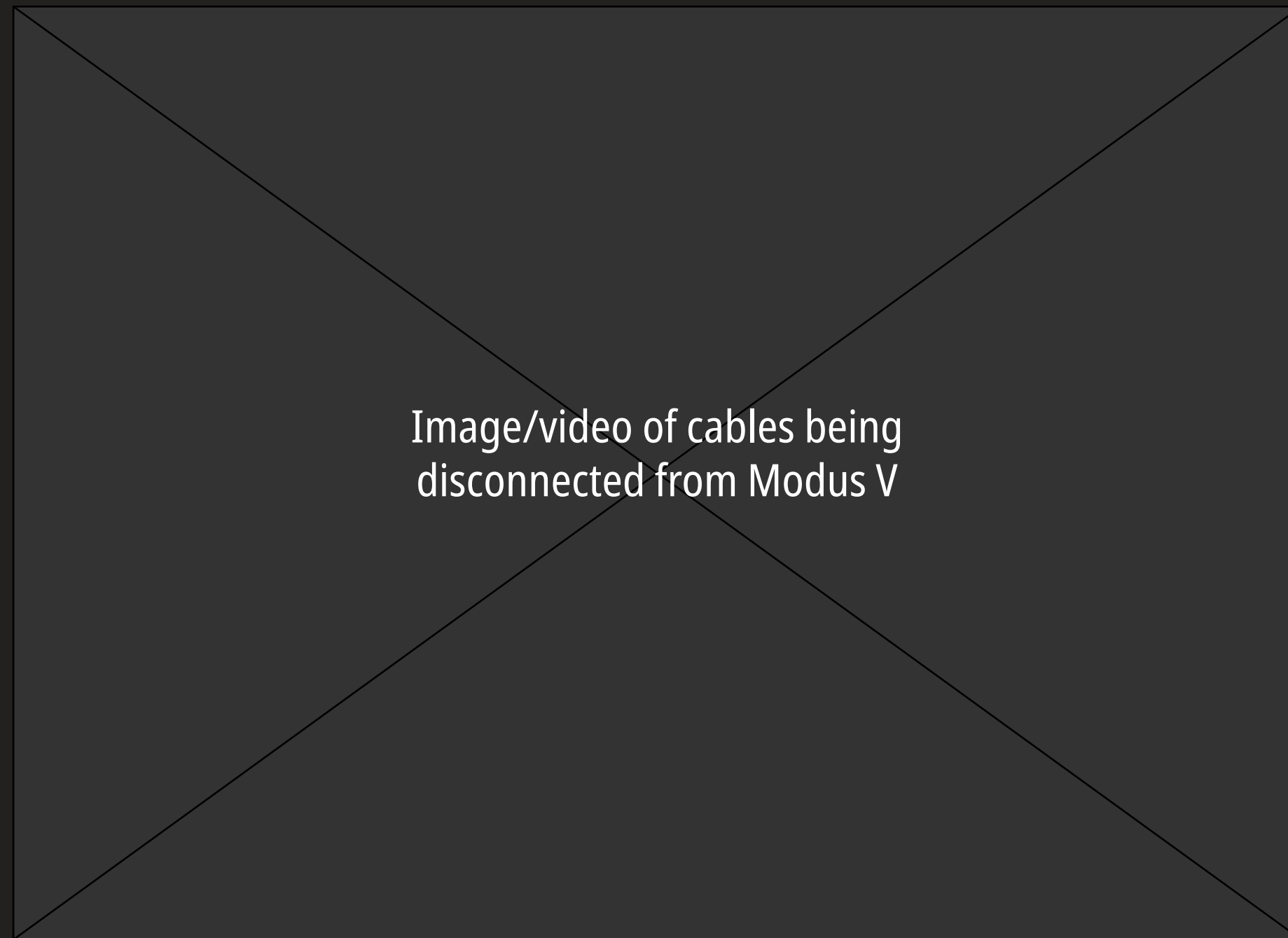

Move



Clean



Shutdown

Move Modus V




Disconnect Cables



Engage Brakes

Engage Modus V brakes.

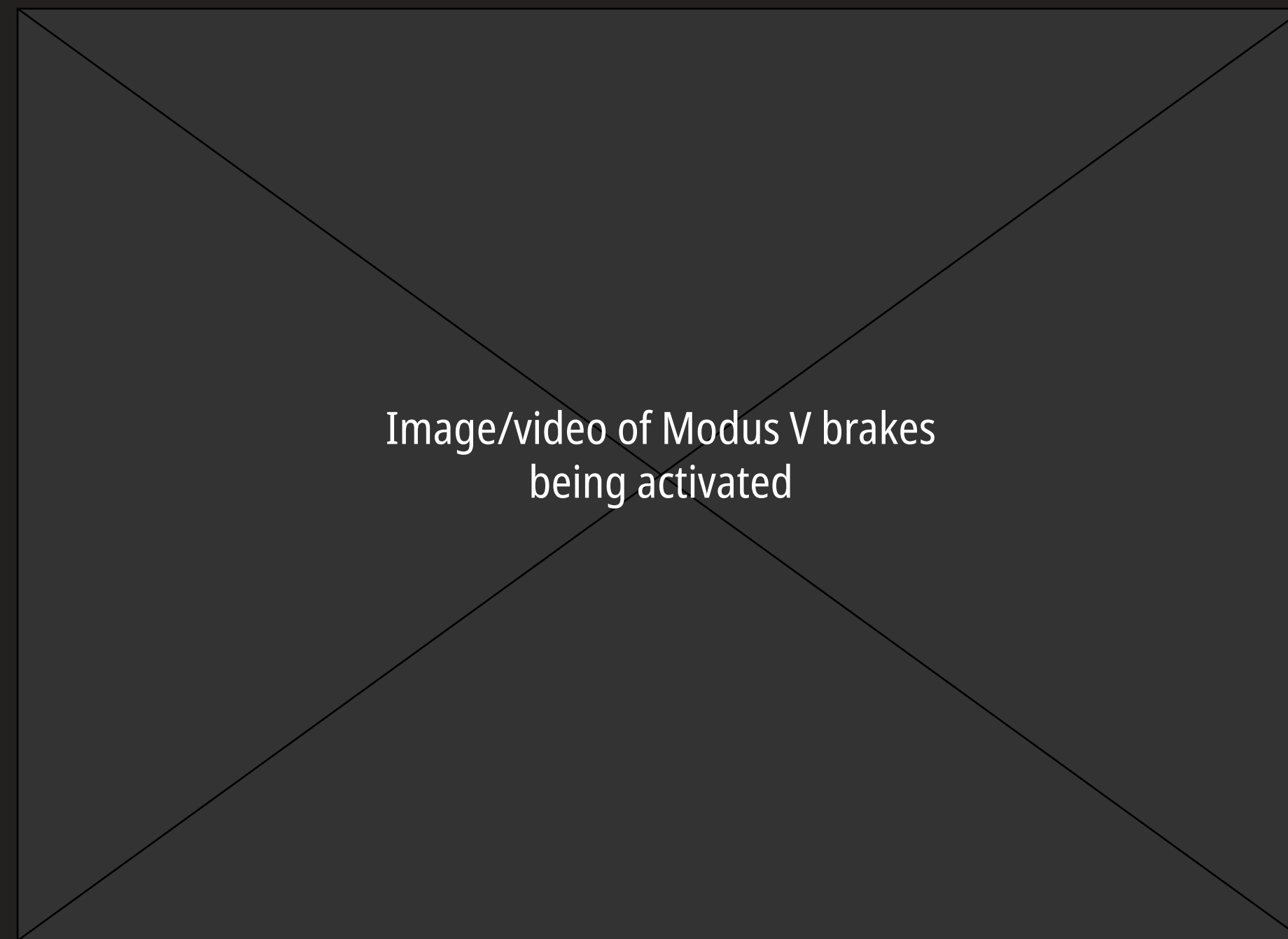

Move



Clean



Shutdown

Tools




Drapes



Cables & Pedal

Disassemble the tools and place them into their cleaning trays.

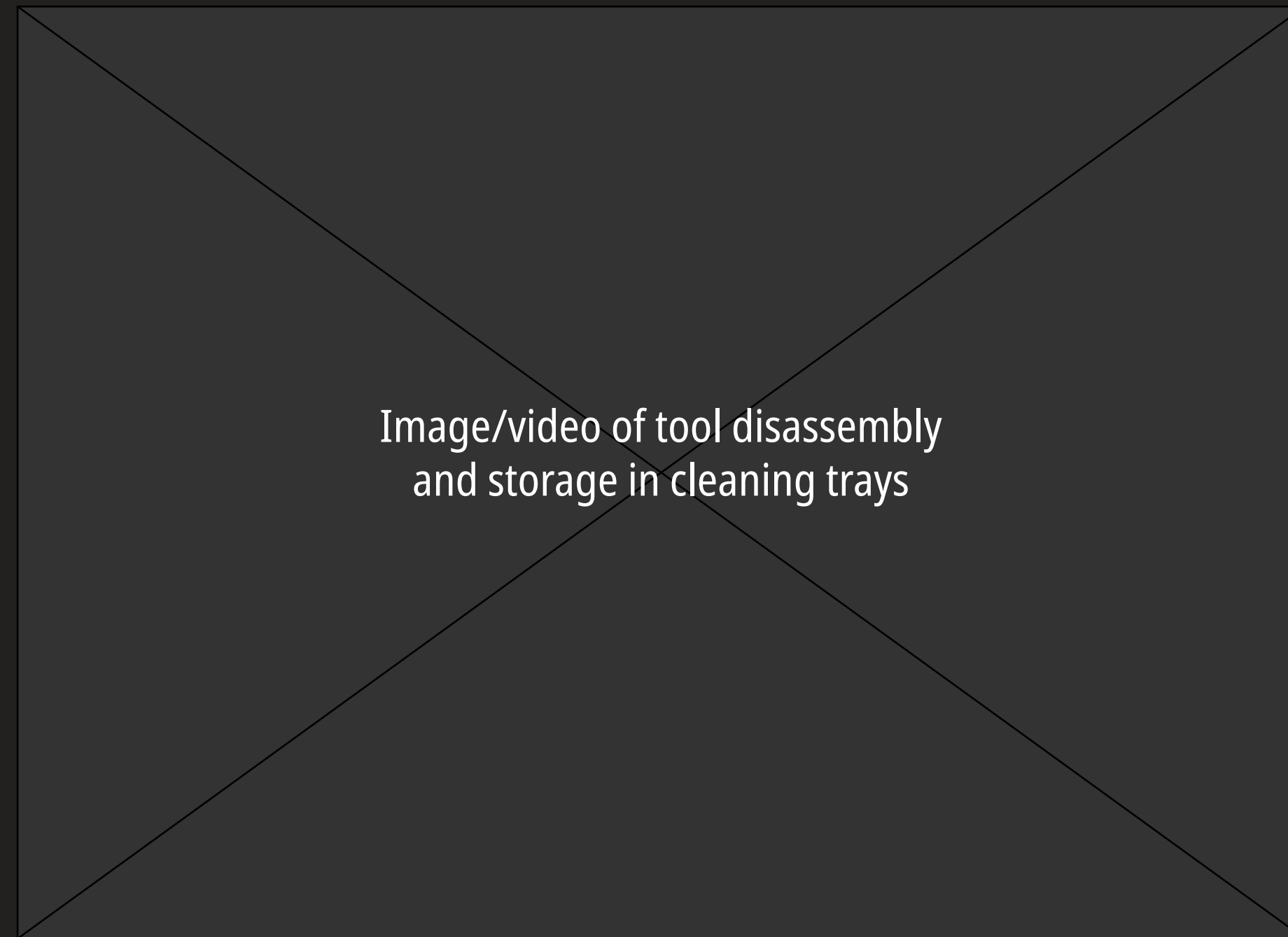

Move



Clean



Shutdown

Tools



Drapes



Cables & Pedal

Remove the drapes and dispose of them.



Move



Clean



Shutdown

Tools




Drapes



Cables & Pedal

Wipe all cables and pedal.

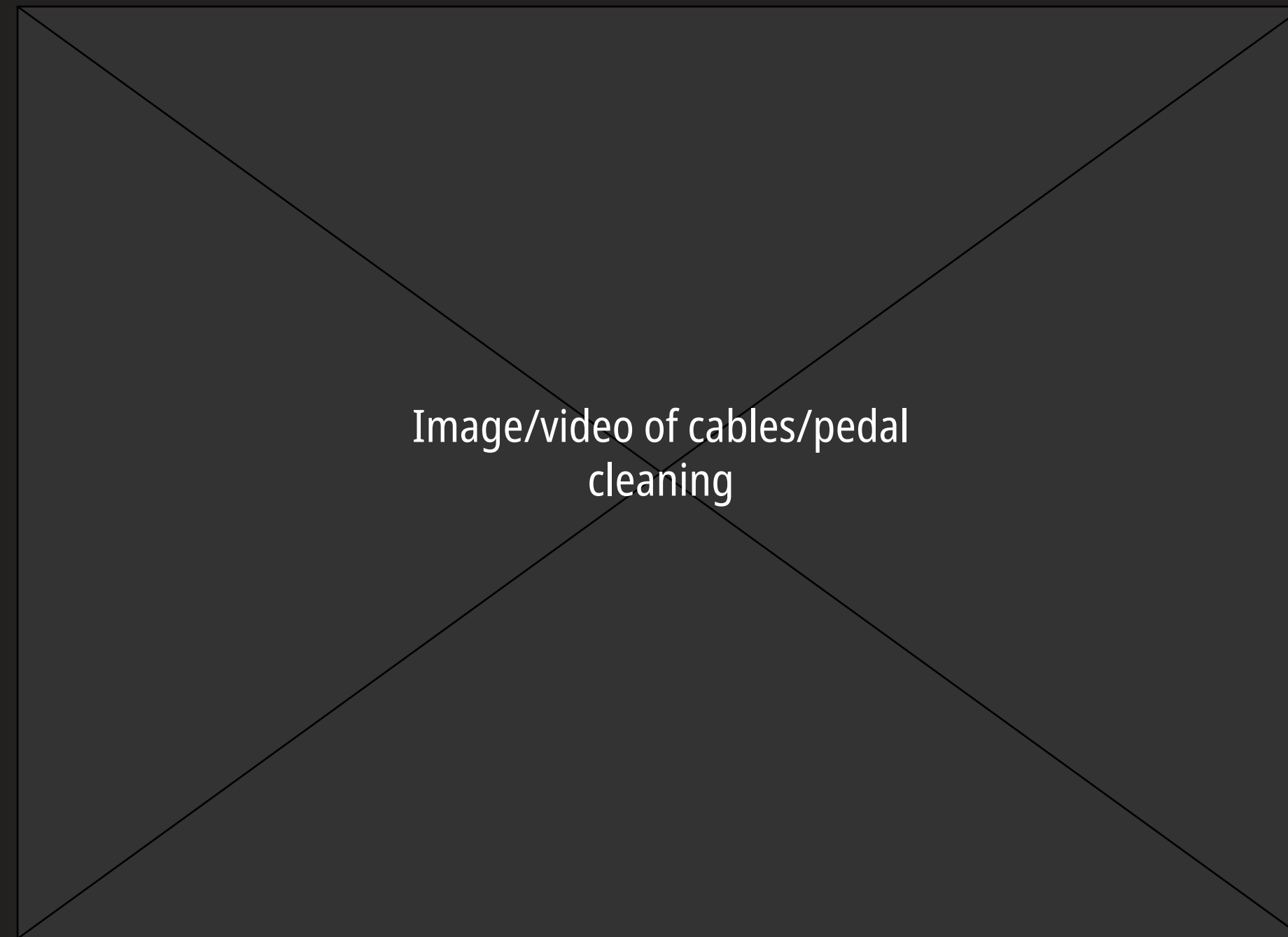

Move



Clean



Shutdown

Store Cables



Store Pedals




Stow



Shut Down

Store the cables.

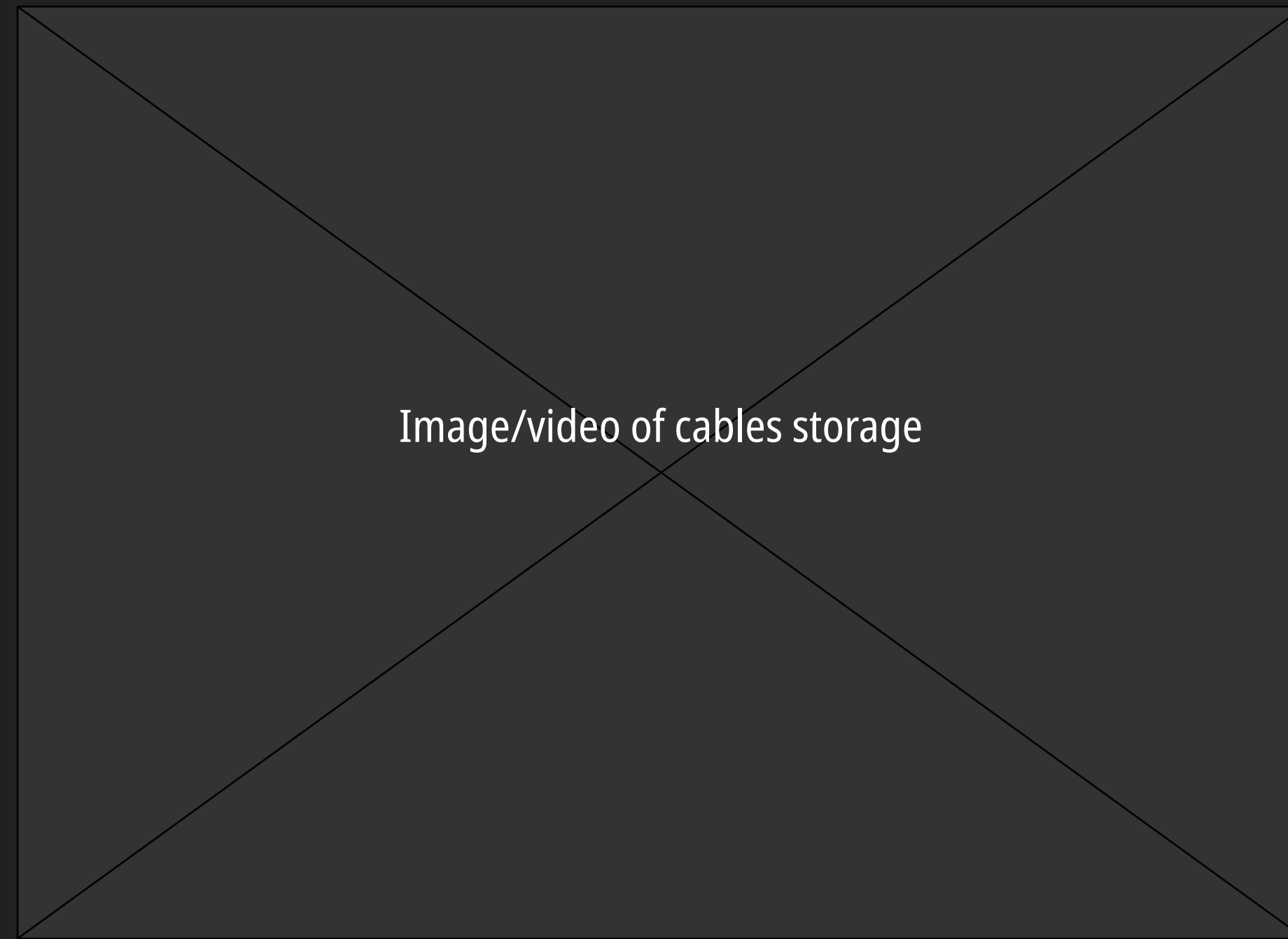



Move



Clean



Shutdown

Store Cables



Store Pedals




Stow



Shut Down

Store the pedal.

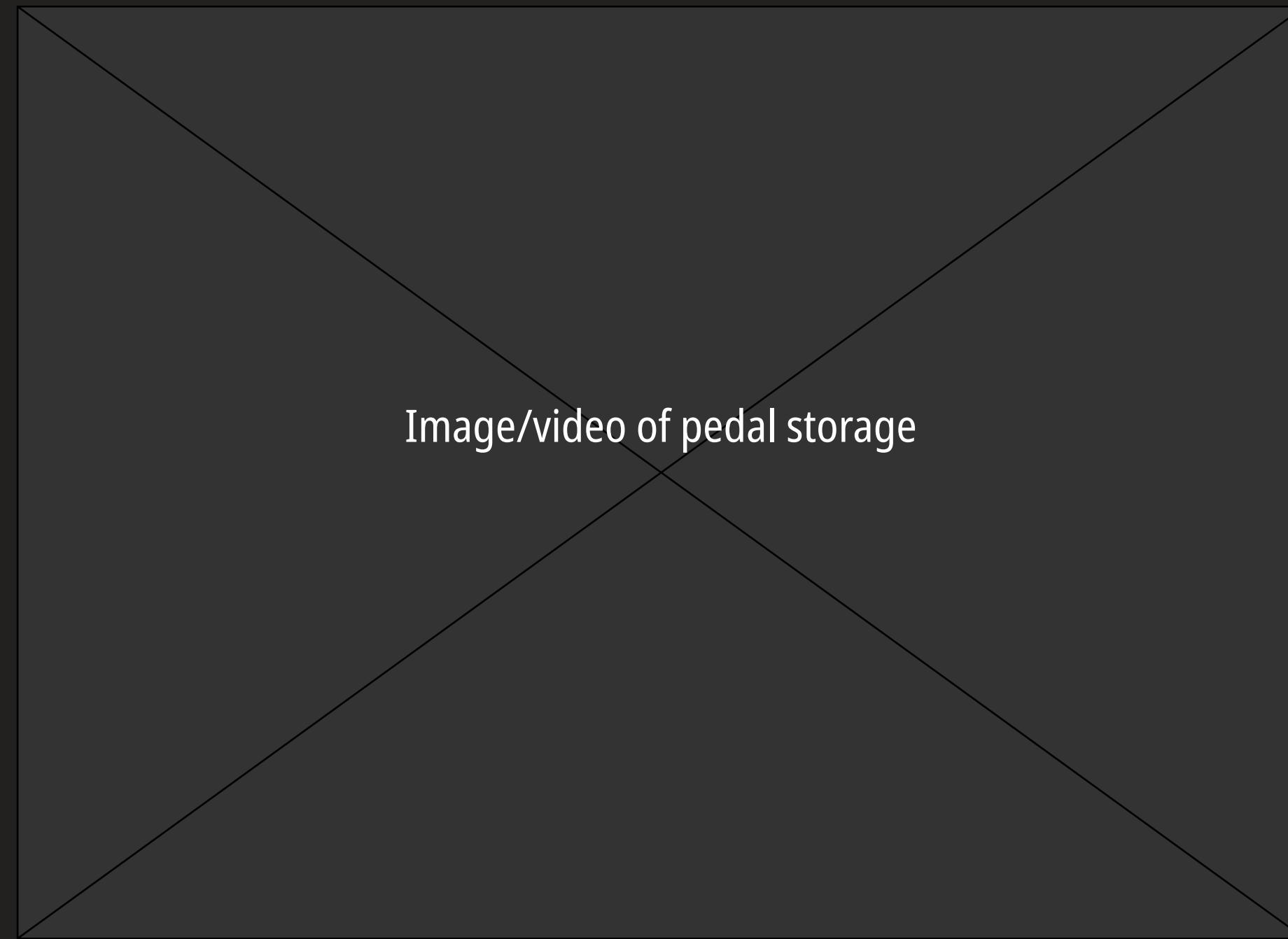

Move



Clean



Shutdown

Store Cables



Store Pedals




Stow



Shut Down

Move Modus V into its stowed position.

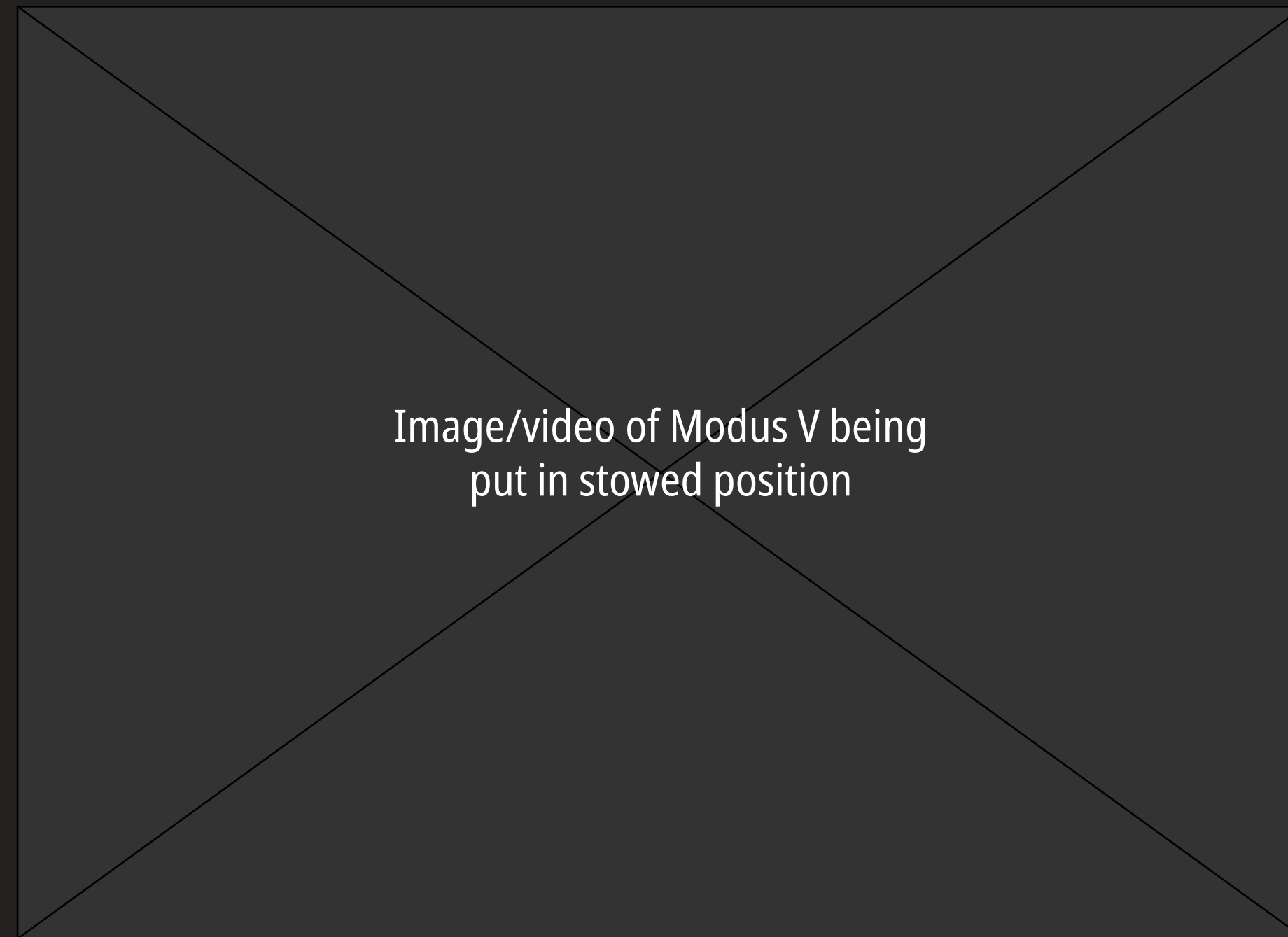

Move



Clean



Shutdown

Store Cables



Store Pedals



Stow



Shut Down

Shut down Modus V.

BACK

